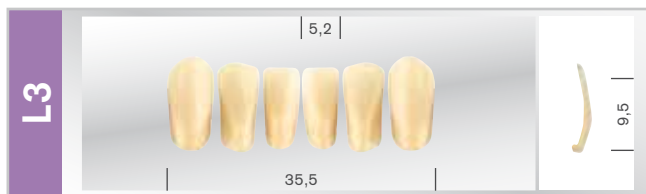
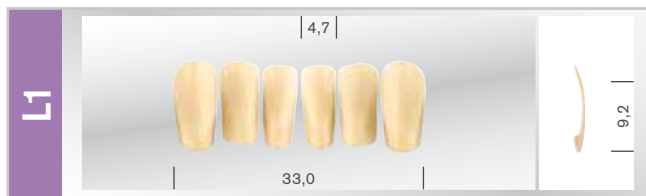
MAJOR AF COMP

AESTHETICS FACETS









LOWER ANTERIORS

WFA POSTERIORIORS

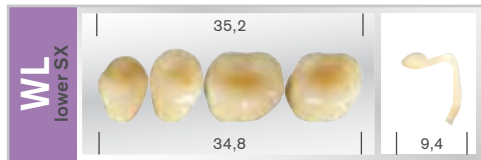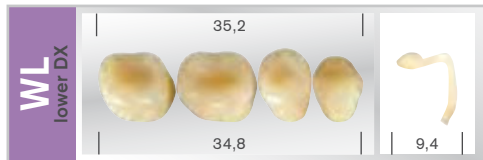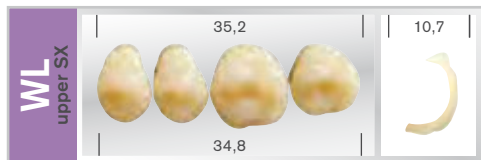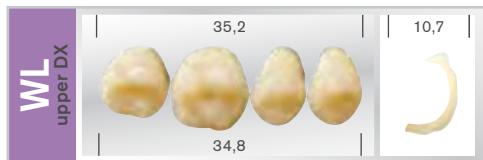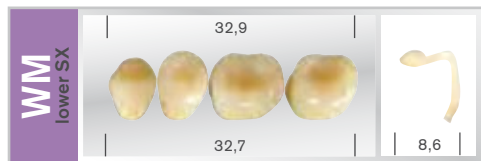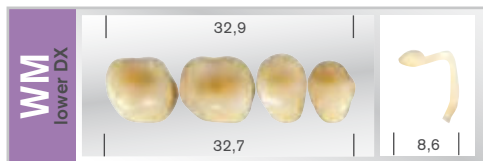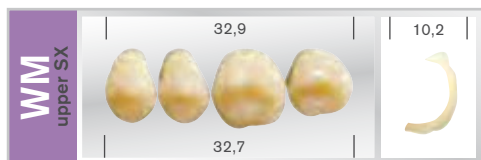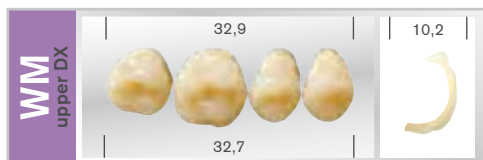WFA Posteriors facets

The new MAJOR AF COMP tooth follows the mounting philosophy according to Prof. Albert Gerber, the palatal cusp of the upper 5th, 6th and 7th tooth occludes in the respective fossa of the lower 5th, 6th and 7th tooth; in the 4th tooth this occlusion is reversed, the vestibular cusp of the lower tooth occludes in the fossa of the upper 4th tooth. The term **WFA (Wide Functional Area)** indicates the type of occlusion done during the mastication, when not only one point of contact is affected but a large functional area of around 2 mm. The occlusal tables respect the concept mortar and pestle. The wide fossa allows a large spectrum of occlusal contacts without interferences. **MAJOR AF COMP** teeth have such an high precision to reduce to the minimum any possible selective grinding made by the operator.

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UPPER ANTERIORS

Anteriors moulds

The anterior teeth include 10 upper and 4 lower moulds, developed according to the most modern aesthetical standards. Available in:

- 10 upper shapes in three different sizes:
 - S (small) - M (medium) - B (large)
- 4 lower shapes L

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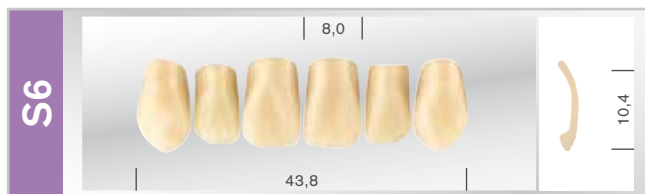
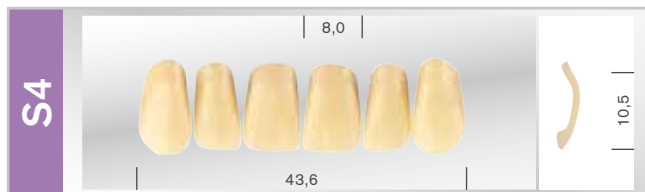
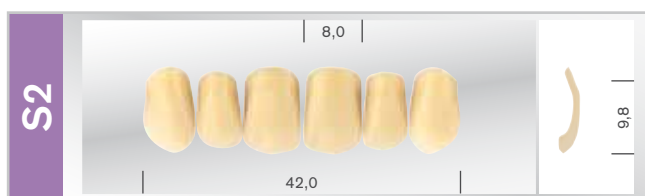
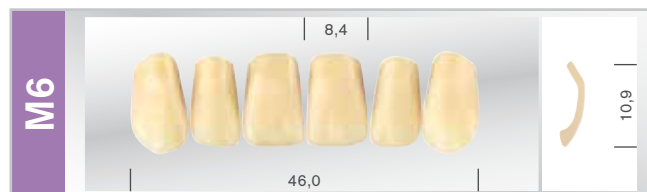
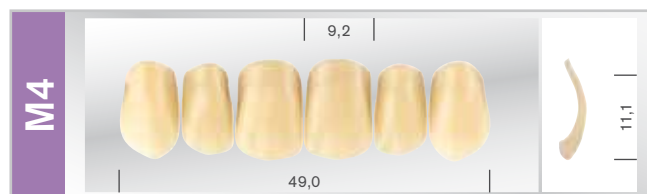
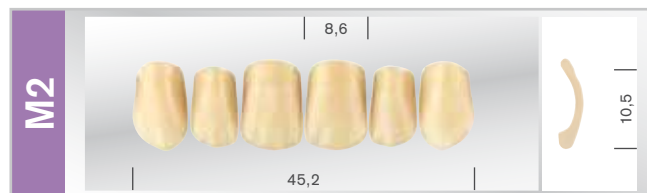
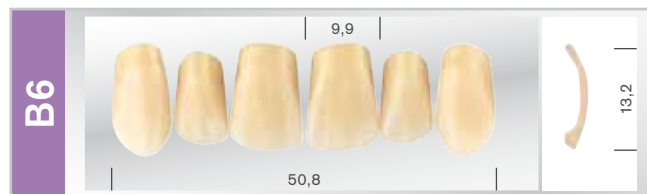
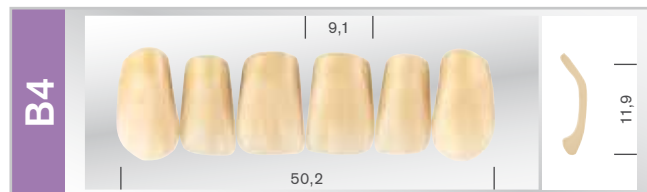
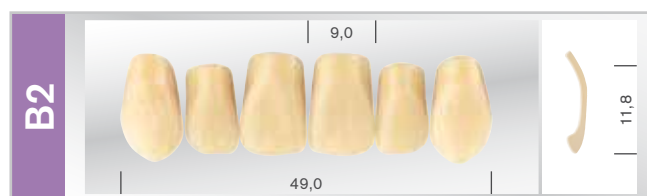
EN SYNTHETIC POLYMER TEETH FOR DENTAL PROSTHESIS

- Store in a clean, dry place away from light.
- To ensure optimum bonding to the denture base resin, grind teeth on the cervix prior to setting up, remove wax and degrease with a methyl methacrylate monomer.
- Direct the patient to adopt daily hygiene protocols, suggest him to avoid excessive brushing which could cause abrasive wear, and not to immerse in alkalis, acids or other substances that can damage acrylic materials.
- This product does not contain substances classified as hazardous conforming to Directive 67/548/EEC and subsequent amendments. Consequently, no Material Safety Data Sheet is available.
- Grinding of the teeth generates dust which is classified as "not otherwise classifiable". Do not inhale. Use appropriate protection. TLV/TWA applies.
- Dispose conforming to local law. Product and packing materials are not listed in the Hazardous Waste Inventory as per C.D. 91/689/EEC 1(4). Patient-contacted materials are at infection risk even for disposal purposes (EU Hazardous Waste Inventory No. 180103).
- Available in the following shade guides: A1-D4.

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MAJOR Prodotti Dentari S.p.A.